

USING YOUR FALL DETECTION PENDANT

- Wear your Fall Detection Pendant around your neck and adjust the lanyard so that it rests at chest level.
- Wear your Fall Detection Pendant button facing forward so that it is easier for you to press in the event of an emergency.
- We recommend that you wear your Fall Detection Pendant inside your clothing to avoid swinging of the pendant.



PENDANT LIGHT

- Pendant Light OFF: In Standby Mode
- Pendant Light **GREEN**: Transmitting Signal to Medical Alarm Device

H9GH; "MCI F": 5 @@'89H97HCB'D9B85BH

- Press or Toss the Fall Detection Pendant forward allowing it to drop to the floor simulating a fall.
- Your Medical Alarm Device will annunciate and transmit a signal to the Emergency Response Center.
- Please wait and tell the operator that you are just testing your system. If you do not explain to the operator that this is a test, emergency help will be dispatched.
- Note: The Fall Detection Pendant takes a few seconds to interpret the movement and determine if an actual fall has occurred. If it determines a fall has occurred, it will send a signal to your Medical Alarm Device, which will initiate the emergency call—just as if you had pressed your button.

The Fall Detection Button does not detect 100% of falls. If you are able, please press your button when you need assistance.